

freeway

SA180C Stand Aid



User Manual

CONTENTS

1.0	Safety Instructions and Warnings.....	4
1.1	Introduction.....	4
1.2	Manufacture.....	4
1.1	European Authorised Representative	4
1.2	Symbols Used	5
1.3	Contraindications / Limitations.....	6
1.4	Intended Use	6
1.5	Warning Notes.....	7
1.6	Operating Environment	7
2.0	Components / Key Parts	8
3.0	Assembly instructions.....	9
3.1	Unpacking.....	9
3.2	Unboxing Stand Aid.....	9
3.3	Electrical connections	10
4.0	Final Inspection	10
5.0	Operating instructions	11
5.1	Manoeuvring the Stand Aid	11
5.2	Raising and Lowering	11
5.3	Open / Close Stand Aid Legs	11
5.4	Emergency Stop Button	12
5.5	Emergency Lowering.....	12
5.5.1	Manual Emergency Lowering	12
5.5.2	Electrical Emergency Lowering.....	12
5.6	Handset Operations	12
5.6.1	Battery Status - Handset.....	13
5.6.2	Service Indication	13
5.7	Battery.....	14
5.7.1	Charging the battery pack	14
5.7.2	Battery Status – Control Box	14
5.8	Battery Control Box Display	15
5.8.1	Service Information read-out:	15
5.8.2	Removing battery pack:.....	16
5.9	Knee pad adjustment	16

5.10	Removing the footplate.....	16
5.11	Adjusting the Upper Assembly	17
5.12	Slings/Belts	17
6.0	Technical Specification.....	19
6.1	SA180C Dimensions.....	19
6.2	Specifications.....	20
6.3	Electrical specifications	20
6.4	Standards Applied	20
7.0	Environmental - Storage and Operating Conditions.....	21
7.1	Normal Operating Conditions	21
7.2	Shipping and Storage Conditions	21
8.0	Disposal.....	21
9.0	Fault Finding.....	22
10.0	General Inspection, Maintenance and Cleaning.....	23
10.1	Service	23
10.2	Inspection.....	23
10.3	Cleaning.....	24
10.3.1	General Cleaning	24
10.3.2	Disinfecting (if necessary)	24
11.0	Warranty	26
12.0	Service Record History.....	27

1.0 SAFETY INSTRUCTIONS AND WARNINGS

1.1 Introduction



Please read and understand this manual in its entirety before using your Freeway SA180C Stand Aid.

The information in this manual is important for the safety of anyone near the Freeway SA180C Stand Aid and must be read and understood to help prevent injuries. It is also crucial to the proper operation and maintenance of the Stand Aid.

The Stand Aid is designed to be used in conjunction with slings and accessories. Please refer to any user guides supplied with these components while reviewing this manual.

Should any questions arise from reviewing this manual, contact your local authorised representative.

Failure to comply with warnings in this manual may result in; injury to the operator and/or client and/or damage to the Stand Aid or related components.

If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

Store this manual with the documents included with the Stand Aid and sling(s). Contents of this manual are subject to change without prior written notice. Ensure you are aware of the sling manufacturer's fitting instructions.



Do not attempt to use this equipment without first understanding the contents of this manual.



Unauthorised modifications on any Prism Medical UK product may affect its safety. The manufacturer will not be held responsible for any accident, incident or deficiencies of performance that occur as a result of any unauthorised modification to its products.



Do not use a clip fit sling on a lifting arm designed specifically for loop fitting slings.

1.2 Manufacture

The Stand Aid is manufactured at the address below:



Prism Medical UK
Unit 1, Tir Llwyd Industrial Estate, St Asaph Avenue, Kinmel Bay, Conwy, LL18 5JZ
Telephone number: 01924 840 100

1.1 European Authorised Representative

The address of the European Authorised Representative for this product:



European Healthcare & Device Solutions (Ireland) Ltd.
Stratton House, Bishopstown Road,
Cork, Ireland.
T12 Y9TC.
Telephone number: +353(86)2280846

1.2 Symbols Used

The Table below includes all Symbols from BS EN ISO 15223-1:2016 that can be found in this Manual and on the Product and what they represent. Refer back to this Table when you are unsure of what a symbol represents.













	Consult instructions before use		Caution – see instructions for use
	Manufacturer	SWL	Safe Working Load represents the maximum load rated for safe operation
	Packaging indicator – This way up		Date of manufacture
	Serial number		Packaging indicator – Keep dry
	Please observe local laws on recycling		For internal use only
	Temperature range	IP_{N₁}N₂	Degree of protection provided by enclosure. N ₁ : Ingress of particles N ₂ : Ingress of water
	Humidity range	MD	Medical Device
	Catalogue number	EC REP	European Authorised Representative

Table 1

1.3 Contraindications / Limitations

There are no known “contraindications” associated with the usage of the Freeway SA180C, provided they are used as per manufacturer’s recommendations and guidelines. However, it is recommended that a client specific assessment is completed by a trained and knowledgeable health care professional to determine the method of transfer.

Freeway Healthcare does not recommend a required number of care givers for the use of our products. This information and recommendation can only be provided after a thorough personalised, case specific assessment, as there are many factors that can influence these decisions.

1.4 Intended Use



For internal use only.

The Freeway SA180C is a Standing Aid to be used by trained personnel. The Stand Aid is a medical device which carries out safe lifting and transfers of an individual from one resting surface to another (such as a shower chair to a bed). Prism Medical UK recommends that the transfer of a patient is fully risk assessed and conducted safely over a short distance only. Carrying out each transfer correctly for minimal strain or risk to the caregiver, while providing complete safety, dignity and comfort to the client. The Freeway SA180C is suitable for patients in the sitting position only, who have a degree of weight-bearing ability but require assistance to stand. By removing the foot tray, The Stand Aid can be used as a walking aid if a competent and trained person completes an adequate risk assessment of the surroundings and the patient is observed during use. The Freeway SA180C is designed to support and promote safe patient handling and transfer for both the patient and carer.

The Stand Aid is one of two components that makes this possible. The other component, the sling, is a specially designed fabric accessory that attaches to the Stand Aid by means of the boom Loop attachment and straps, and holds the patient during operation.

Please refer to the user guides supplied with the sling and reference them while reviewing this manual.

The functions of raising and lowering the boom, and opening/closing the legs are accomplished by pressing buttons on the hand control.



The device is used under instruction and the operation of the aid is undertaken by a trained carer. A risk assessment must be performed before using any other manufactured slings or Stand Aid to ensure safe use can be established.

- The Freeway SA180C is intended to be installed on a flat and levelled surface prior to use.
- The Stand Aid must be installed only by persons authorized by Prism Medical UK or who have the rights to install and commission the Stand Aid safe for use.
- Under no circumstance should the Freeway SA180C or the entire system be put in control of a person who has not been properly trained in the use and care of this equipment. Failure to adhere to this warning may result in serious injury to the operator, and / or the individual being transferred.
- In facilities where more than one operator will be responsible for using the Stand Aid and sling(s) it is imperative that all such members be trained in the Freeway SA180C proper use. A training program should be established by the facility to acquaint new operators with this equipment.
- The Freeway SA180C, and associated slings are not toys. Do not use it for unsafe practices. Do not allow children to play with the Stand Aid or any of its components.
- Your guarantee is void if persons unauthorised by Prism Medical UK perform work on the Stand Aid.

- To maintain optimum function, the Freeway SA180C should be inspected and maintained on a regular basis. See section 'General Inspection, Maintenance and Cleaning' within this user manual.
- Any accessories used with this Stand Aid including belt(s), should be checked to ensure that they are in good working order. Check for signs of wear to each component prior to use. Report any unusual wear, or damage immediately to your local authorised dealer.
- The Freeway SA180C and Slings(s) are intended only for transferring of a person. Prism Medical UK will not be responsible for any damage caused by the misuse, neglect or purposeful destruction of the Unit, and/or its associated components.
- The Stand Aid and its associated parts are certified to a maximum load of 180kg. Do not exceed the maximum rated load of any of the components.
- Ensure that a clear space is maintained around the Stand Aid. Before performing a transfer check for and move all obstacles out of the way.
- Your Product is for Aiding a human to transfer. Do not use it, or allow it to be used, for any other purpose.
- Protecting the people present, visually monitor the sling(s) loops connection points during transfer stages, so the belt remains firmly attached to the Stand Aid
- In areas where children are prone to be present be vigilant when carrying out a transfer.
- To reduce the risk of unintended use, when the Stand Aid is not in use remove the sling(s) from the product to prevent entrapment or strangulation should the device be tampered with.
- Between Stand Aid and Body-Support Unit, the lowest maximum load shall always be used.



You may need to seek specialist advice on how to assist some people with specific moving and handling needs. Sources of advice include, but is not limited to, professional bodies and organisations, occupational therapist, physiotherapists, manual handling advisers and ergonomist with experience in health and social care.

1.5 Warning Notes



Your Freeway SA180C has been manufactured and tested to exceed BS EN 10535:2006. This does not mean that it can be used without care. ALL OPERATORS should have read the operating instructions and appreciate this warning section.

1. ALL Stand Aids are less stable on sloping surfaces. A 5-degree slope is the maximum permitted and then only with great care.
2. ALL Stand Aids are dangerous to the person being transferred when used recklessly or pushed at speed.

1.6 Operating Environment

The Freeway SA180C is suitable for use within the professional health care facility environment as well as the home health care environment. The Stand Aid is not suitable for any special environments.

The Stand Aid is not intended to be used in environments where there are rapid changes in the environmental temperature and humidity during intended use.

2.0 COMPONENTS / KEY PARTS

Please see below to familiarise yourself with the components of the Freeway SA180C Stand Aid. The images below show the contents of the Stand Aid. If you have not received all the components, contact your Prism Medical UK dealer immediately – contact details are provided on the last page of this manual.

Item	Description	Item	Description
1	Optional Body Protector	12	Knee Pad
2	Looped Sling Attachment	13	Electric Emergency Lowering Button
3	User Handle	14	Leg Strap Anchor Loop
4	Lifting Arm	15	Knee Pad Adjustment
5	Lifting Arm Adjustment	16	Optional Calf Strap
6	Manual E-lower	17	Electric Leg Actuator
7	Handset	18	Foot Tray
8	Lift Actuator	19	Foot tray Toe Pad
9	Battery	20	Front Casters
10	Emergency Stop button	21	Rear braked Casters
11	Product Serial number label	22	Handle Bar

Table 2

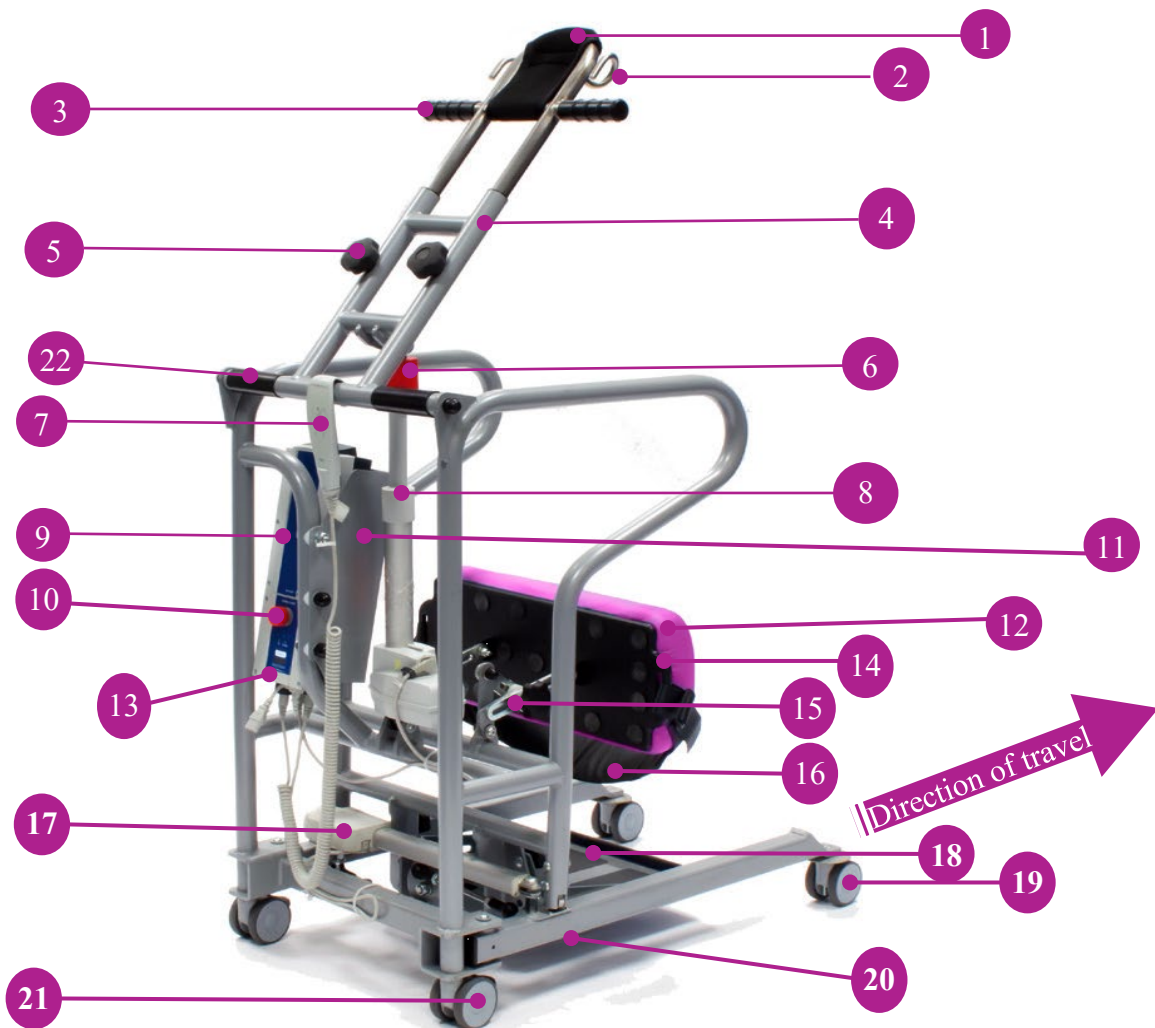


Figure 1

3.0 ASSEMBLY INSTRUCTIONS

3.1 Unpacking



Some of the parts are heavy and will need to be lifted with care. Items weighing over 20kg will require a two person lift. Please refer to (6.0) technical specifications for components weights.



The Stand Aid will arrive to you in a robust box, please be careful when removing the components from the box. Please read the user guide in full before operating.

This user manual should be kept safe for future reference.

The Stand Aid has been specifically designed to be installed in both the professional and home health care environments.

No matter the environment, health and safety factors should be considered to ensure the safety and essential performance of the Stand Aid and to avoid unnecessary damage or injuries to people within the area of the Stand Aid.

The environment in which the Stand Aid is situated in, whilst carrying out a transfer, is required to be on a flat surface with no steep inclines.

3.2 Unboxing Stand Aid



When using a sharp knife, be careful not to damage the product.

This section will summarize the layout of the Stand Aid Packaging and what is included in the Box. It is recommended a knife is used for smoother unpacking of the Stand Aid. The Stand Aid is packed into a single box (875x620x1000) weighing approximately 55kg.

Using a knife to open the box around the perimeter, the box should open, remove the internal packaging to access the product. It will include all the following components. It is recommended that the components are removed in the numerical order below.

1. Full Assembly of the Freeway SA180C
2. Handset
3. Charging Lead
4. User Manual

3.3 Electrical connections



Connect all the cables to the control box as shown in the pictures opposite.

- A. Connect the lift actuator cable connector to the second port (marked No. 1) from the left side of the control box (as shown in Fig. 2).
- B. Connect the leg spreading actuator cable connector to the third port (marked No. 2 in Fig. 2)
- C. Connect the handset connector to the large port on the left side of the control box. A locator pin on the plug ensures correct orientation of the hand control connector (as shown in Fig. 4).
- D. Connect the mains cable/power cord to the control box (as shown in Fig. 5).



Figure 2



Figure 3



Figure 4



Figure 5



Ensure that all cables are located and secure before operating the Stand Aid.

4.0 FINAL INSPECTION

Before first operation of the Stand Aid:

- Confirm all cables are located and secure.
- Ensure the red emergency stop button on the control box is deactivated. (Rotate clockwise to release if required- Refer to section 5.4 for guidance).
- Press the up button on the handset and confirm the actuator raises the boom.
- Press the down button on the handset and confirm the actuator lowers the boom.
- Press the button on the handset to operate the leg opening and confirm the actuator moves the legs outwards.
- Press the button on the handset to operate the leg closing and confirm the actuator moves the legs inwards.
- Press the red emergency stop button on the control box and check that actuators do not operate until the button is reset. (See section 5.4 for guidance).
- Check the manual emergency lowering function works properly. (See section 5.5 for guidance)
- Check the rear castors brakes function properly.
- Check the battery pack is fully charged.

Your Stand Aid is now ready to use.

5.0 OPERATING INSTRUCTIONS



The Stand Aid may be heavy for users and will need to be lifted with care, please operate the device with the use of the castors wherever possible to prevent any injury.

Appropriate training in lifting and handling procedures should be undergone by any person operating a Stand Aid, for their own and the client's safety and comfort. These instructions are designed to cover the method of using the Stand Aid.

5.1 Manoeuvring the Stand Aid

- A. To move the Stand Aid forward, hold onto the handle bar and push forward.
- B. Do not stand on the leg actuator, or use the actuator to push the Stand Aid.
- C. When turning the Stand Aid, use both hands on the handle. Alternatively, it is possible to rotate the Stand Aid by applying the brake to a single rear castor and rotating the Stand Aid about the braked castor. This movement should be performed with a smooth, slow action to avoid swinging the patient unnecessarily.
- D. The Freeway SA180C Stand Aid has two rear castors with brake.
- E. The rear castors can be braked for rotation (by applying a single brake), lateral movement, and parking. To apply the brake, press the brake pedal down with your foot (as shown in Fig. 6). To release the brake, press the raised pedal towards the wheel (as shown in Fig. 7).
- F. During lifting, the rear wheels should remain unlocked so that the Stand Aid will not move.



The wheels should be locked if there is a risk of the Stand Aid moving to the patient. For example, when lifting the patient from the floor.



Figure 6



Figure 7

5.2 Raising and Lowering

The up and down movement of the lifting arm on the Freeway SA180C Stand Aid is achieved by an electric actuator which is controlled by hand control. The hand control has two buttons with directional arrows up and down. The actuator stops automatically at the limit of travel in both directions.

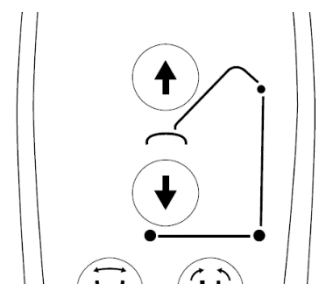


Figure 8

5.3 Open / Close Stand Aid Legs

The legs of Freeway SA180C Stand Aid are opened/closed using the appropriate buttons on the hand control, see figure 9 for guidance. The legs' motion will be stopped whenever the hand control button is released. The legs can be opened to enable access around arm chairs or wheel chairs. When relocating the Stand Aid, manoeuvring through narrow doorways and/or passages, the Stand Aid legs should be in the closed position.

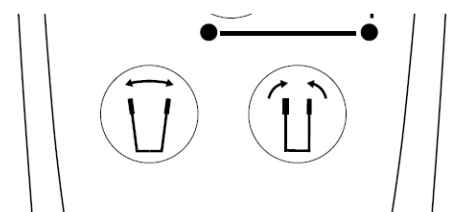


Figure 9

5.4 Emergency Stop Button

The Freeway SA180C Stand Aid is fitted with an Emergency Stop button.

Push the Emergency Stop button (as shown in Fig. 10) to cut all power on the Stand Aid.

To resume power, release the emergency stop button by turning in a clockwise direction (indicated by arrows on the Emergency Stop button) as shown in Fig. 11.



Figure 10



Figure 11

5.5 Emergency Lowering

5.5.1 Manual Emergency Lowering

In case of power failure, it is possible to mechanically lower a patient placed in Freeway SA180C Stand Aid.

Turn the manual lowering handle in clockwise direction (as shown in Fig. 12) to lower the boom until the boom reaches a safe position.



Figure 12



The manual emergency lowering system should be used only if the lowering procedures described in the previous section of the manual do not work. Should you have any concerns or questions contact your local authorised Prism Medical UK Representative.



Do not use the Stand Aid after the manual lowering mechanism has been used. The lift must be reset by a qualified technician after use. Contact your local authorised Prism Medical UK Service Provider.

5.5.2 Electrical Emergency Lowering

The Stand Aid lifting arm can be raised and lowered by pushing the emergency lifting and lowering buttons on the control box with a suitable object, such as a pen nib or pencil (as shown in Fig. 13).



Figure 13

5.6 Handset Operations

The Hand Control has 4 functions: Up and Down, this will raise and lower the Stand Aid lifting arm, and Legs Open and Legs Close, these functions will open and close the legs.

Press down on each symbol to operate the desired function. It is not possible to use two functions at the same time. Once the Stand Aid has reached the extent of its travel in a given direction, the actuator will automatically stop, allowing you to then choose the opposite direction.

The handset is attached to the control box via a flexible, coiled cable that is secured in place with a friction-fit plug. The coiled cable is designed to give the greatest number of options for carer positioning without having a trailing cable around the patient. The handset also incorporates a hook which gives the carer flexibility whilst moving/positioning the patient. Clear and easy to understand labelling of the buttons enable ease of use for the care giver.

The handset also displays battery level and service requirement by the use of LED lights.



Figure 14

5.6.1 Battery Status - Handset


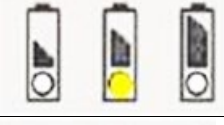


Battery State	Indication	Comment
1		The battery is OK - no need for charging (100-50%) the third LED is green
2		The battery needs to be charged (50-25%) the second LED is yellow
3		The battery needs to be charged (less than 25%) The buzzer sounds when a button is pressed The first LED is yellow
4		The battery needs to be charged. Some of the functionality of the lift is lost and it is only possible to drive the lifting arm down. By continuous use the diode blinks and starts to give an audio signal

Table 3

5.6.2 Service Indication

The service indicator (Yellow LED) will blink when it is time for service. The standard setting is every 12 months/ 8000 cycles, whichever comes first.



Indication	Comment
	When it is time for service, the service indicator will remain illuminated 2 minutes after use. Afterwards the control box will power down to save the battery.
	When the service indicator lights up the system will still be functional.

Table 4

5.7 Battery



The battery pack should never be stored for long periods of time without a regular charge.

The battery pack is protected encased unit located on top of the control box, on the rear of the product. The control box, below the battery, will inform the user when the battery is flat and requires charging, by an illuminated LED. An LED illuminate when the battery needs recharging. Complete the lift and place the battery on charge.

5.7.1 Charging the battery pack

The battery pack can be recharged via mains lead (terminated with a figure of eight plug)

- A. Fit the mains lead figure of eight plug into charger link cable (see Fig. 15).
- B. Plug the 3-pin mains plug into a suitable mains outlet and switch the mains supply on.
- C. Charging is automatic and will normally take eight to twelve hours to fully charge (from completely discharged state). You cannot overcharge the battery pack if left for longer periods of time.
- D. To return the Stand Aid to use, switch off the mains supply. Remove the figure of eight plug from the socket at the base of the control box.
- E. The Stand Aid is now ready for use.



Figure 15

5.7.2 Battery Status - Control Box

The Table below provides information on the four battery states seen on the Control Box LCD. Refer to this table when unsure of what the symbols represent.




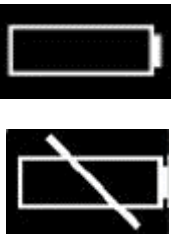
Battery state	Symbol	Comment
1		The battery is ok, no need for charging (100 - 50 %)
2		Battery needs charging (50 -25%)
3		Battery needs charging. (Less than 25%) Buzzer sound is provided when a button is pressed in this battery state.
4		The battery needs charging. At this stage some of the functionality of the lift is lost. At this battery stage, it is not possible to drive the lifting arm up or down. Furthermore, an audio signal will sound when a control button is activated (17V or lower). The symbol will switch between the two pictures for 10 seconds. The battery symbol is shown when the box is active until power down (2 minutes after use).

Table 5

5.8 Battery Control Box Display

The Table below provides information on all Symbols seen on the Control Box display and what they represent. Refer to this table when unsure of a Symbol displayed on the Control Box.

Control box feature	Symbol	Comment
Charging of battery		When the mains cable is plugged in and a control button is activated the symbol to the left is shown on the display until power down 2 minutes later. The purpose if the symbol is to inform the user that it is not possible to use the lift when it is plugged into the mains.
Short Circuit		If there is a short circuit, the control box will show the short circuit symbol with a recommendation to check the connections. This symbol will be shown until the short circuit has been repaired.
Service		The control box will show the service symbol when it is time for service. The standard settings is after 12 months/8000 cycles. After each power down, first time that the symbol (100 milli-seconds) so that the user gets a reminder about checking the display. The 'SERVICE' Text will blink 3 times, then a static service symbol will be shown (10 seconds in total). Even though it is time for service the system will still be functional and work as normal.
Overload channel 1 only:		When Overload occurs (according to the pre-defined current cut-off limit) the overload symbol will be shown on the display. The 'MAX. Text will blink 3 times and the overload symbol will be shown for 10 seconds in total.

Table 6

5.8.1 Service Information read-out:

Basic service information can be read out on the display. To display the service information on the display press the lifting arm up button (Only for ½ second press). The information (below) will be shown for ½ minute or until other buttons are activated.

If “No Days” are chosen for service interval then the display will show Days since last service.

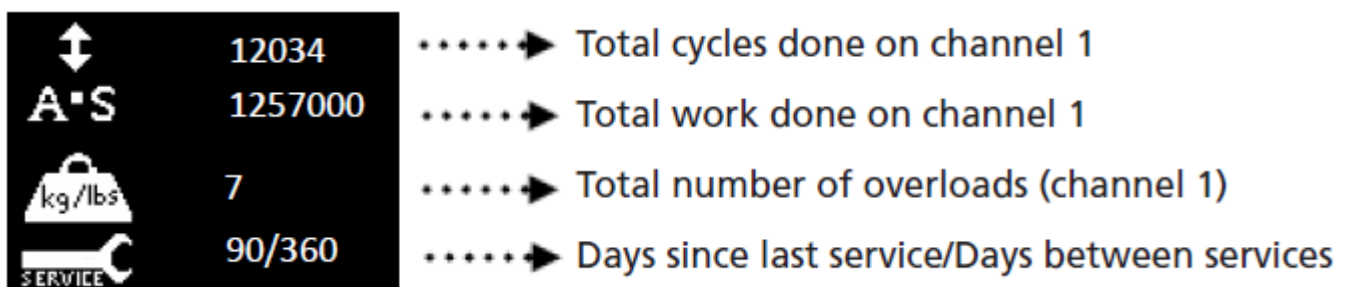


Figure 16

5.8.2 Removing battery pack:

The battery pack has an option to remove the battery pack for replacement or servicing requirements.

- Take your hand and grab onto the top of the battery by the use of its handle.
- Directly underneath the handle you will find a lever. Using two fingers pull back on the lever to release it from the control box bracket (Fig. 17).
- Now the battery is released you can simply lift the battery off the control box by lifting vertically.



Figure 17



Figure 18

5.9 Knee pad adjustment

The knee pad can be adjusted for reach, height and angle. Adjustment is made by loosening the two star knobs either side of the knee pad (as shown in Fig. 19) and position the knee pad (as shown in Fig. 20).

Once the knee pad has been positioned correctly tighten the star knobs and ensure the knee pad is secure

The knee pad includes loops for the optional knee strap kit. The strap should be passed through the loop at either side of the pad (as shown in Fig. 21) and fastened.



Figure 19



Figure 20



Figure 21



Ensure the adjustment star knobs are tight on the knee pad to prevent knee support movement during use.

5.10 Removing the footplate

The footplate can be removed by loosening and removing the two star knobs (as shown in Fig. 22).

With the two star knobs and bolts removed, carefully lift the foot plate from the frame (as shown in Fig. 23). It is recommended to refit the star knobs to the frame for future use.

When re-attaching the foot plate, ensure it is securely fitted before use (as shown in Fig. 24).



Figure 22



Figure 23



Figure 24



When using the equipment to assist a person to stand onto the floor (with the footplate removed), the brakes must be applied.

5.11 Adjusting the Upper Assembly

The upper assembly can be adjusted for reach, height and angle. Adjustment is made by loosening the two star handles by **3 turns in an anticlockwise direction** (Figure 25). From here the upper assembly can be adjusted back and forth to its preferred position. Once the position is chosen, the upper assembly can be fixed in place by tightening the star handles until **sufficiently tight in a clockwise direction** (Figure 26).



Always ensure that before every use, the star handles are tight. If they are not, tighten accordingly.

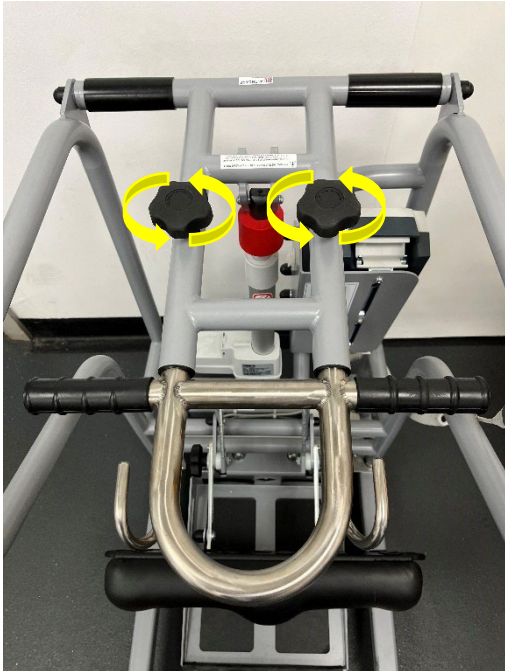


Figure 25

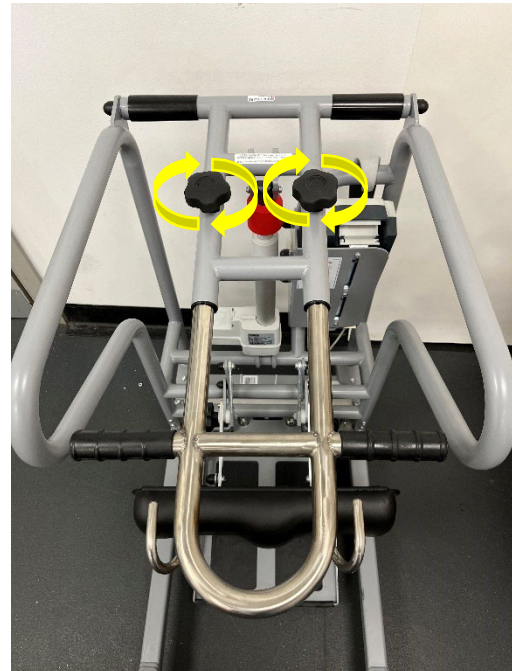


Figure 26

5.12 Slings/Belts

Sling loops should be attached as follows:

1. Put the required sling loop onto your finger and thumb and open the selected sling loop up. (Figure 27)
2. Slide the sling loop onto the sling hook. (Figure 28)
3. Follow the hook profile around until the sling loop is positioned at the base of the hook. (Figure 29)



Figure 27



Figure 28

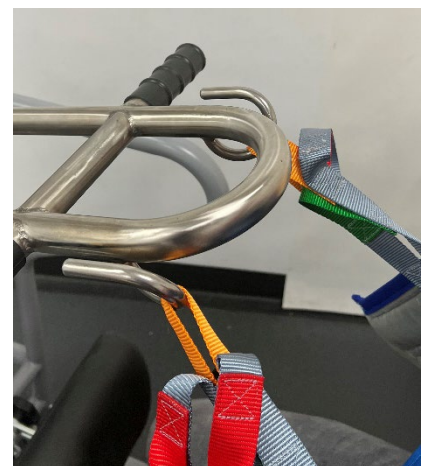


Figure 29

To remove the sling, simply reverse the processes shown in the above statements.



Ensure you select the correct sling size for the person being transferred. The sling should be fitted around the person's stomach/lower back and not around the chest. (Figure 30). Correct fitting of the Sling is important for the patient's safety and to maintain a good posture. (Figure 31).



Figure 30



Figure 31

Prism Medical UK recommend the use of the Freeway Thoracic Support Sling (type 'B' applied part) to be utilised with the Freeway SA180C Stand Aid. It is at the user's discretion to use alternative supplied product. In utilising another manufacturer's sling, checks must first be made to ensure the sling is safe to use and meets the requirements of BS EN ISO 10535 before its use.

The slings with a safe working load of 180kg that can be used with the Freeway SA180C stand aid are shown below in Table 3, complete with product codes. For all components, the lowest maximum safe working load must always be adhered to.

Size	Freeway Sling Range - Product Codes
Freeway Thoracic Sling	
Small	12018Y0500
Medium	12018Y0400
Large	12018Y0300
XL	12018Y0200

Table 7

6.0 TECHNICAL SPECIFICATION

6.1 SA180C Dimensions

All dimensions below are shown in millimetres.

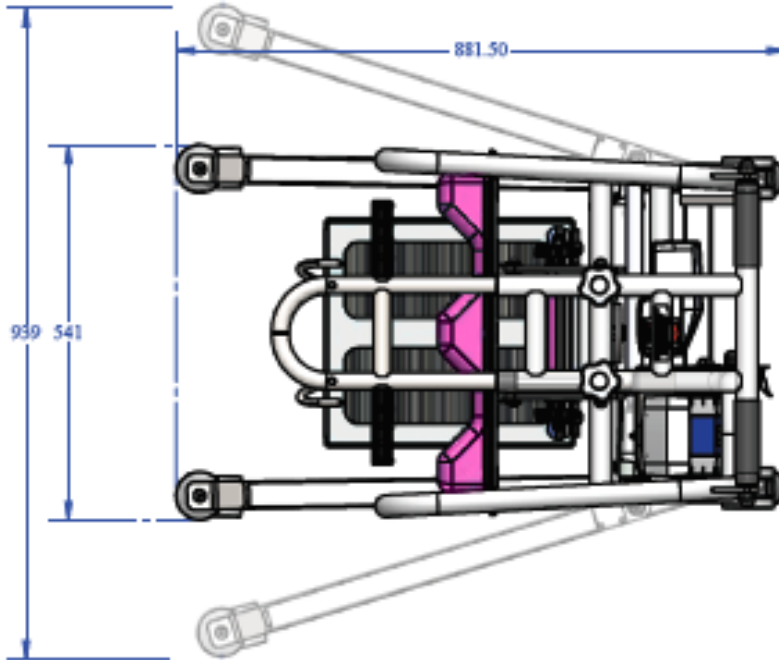


Figure 32

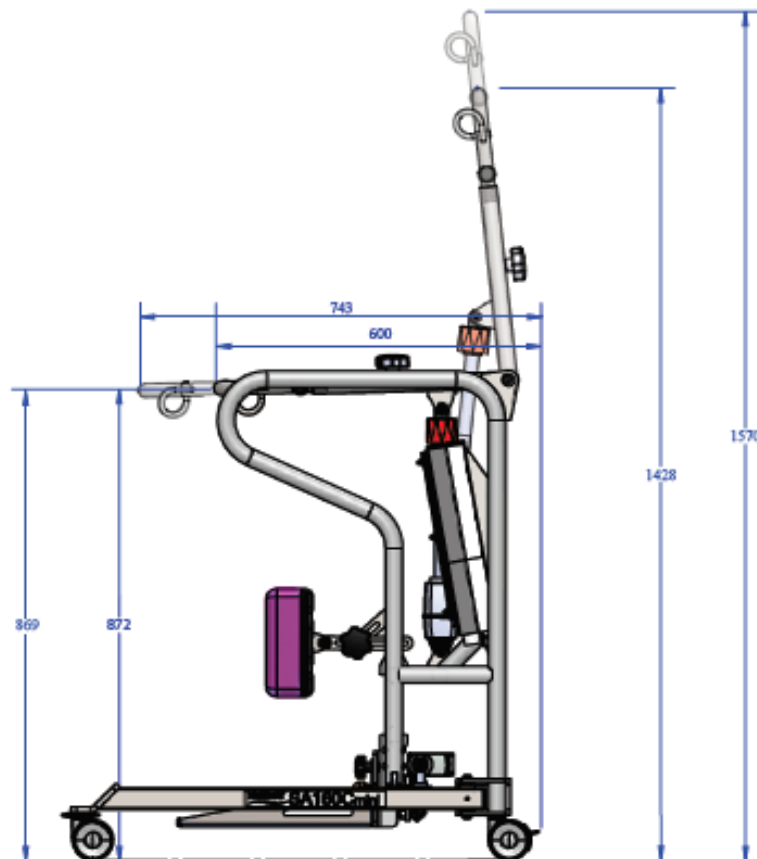


Figure 33

6.2 Specifications

Dimensions	
Turning Circle	915mm
Overall Height	1430mm
Overall Width	530mm
Ground Clearance	50mm
Front and Rear Castors Size	75mm

Table 8

Weights:	
Safe Working Load (SWL)	180 kg
Full Product	52kg

Table 9

Operational Forces	
Moving Force (with 180 kg)	105 N
Moving Force (with 0 kg)	20 N
Operation of the Brake	35 N
Handset	4.5 N
E-Stop Button	12 N

Table 10

6.3 Electrical specifications

Battery type:	2 x 12-volt Rechargeable sealed lead acid.
Battery capacity:	24 V DC, 1 x 2.9 Ah
Charger rated input:	230VAC 50/60Hz
Charger rated output:	27.4/29.0 VDC @ 0.8A

Electric Shock Protection

Charger:	Class II *Lift - internal power source
Degree of shock protection:	Type B *Lift - internal power source
Duty cycle:	10% Use – 90% Rest (90 seconds use - 810 seconds rest)
IP rating, control box:	IPX4
IP rating, battery:	IPX5
IP rating, handset:	IPX4

Expected Product Lifetime

10 yrs. depending usage and compliance to maintenance, servicing and LOLER inspections.

6.4 Standards Applied

The standards that have been applied to the device are as follows:

- EN 10535:2006 Stand Aids for the transfer of disabled persons. Requirement and test methods

7.0 ENVIRONMENTAL - STORAGE AND OPERATING CONDITIONS

The SA180C is intended for internal use within normal environmental conditions.



It is not intended to be used in environments where there are rapid changes in the environmental temperature and humidity during intended use.

7.1 Normal Operating Conditions

+5°C to +40°C (41°F to 104°F) at a relative humidity between 15% to 90% RH, non-condensing but not requiring a water vapour pressure greater than 50hPa and atmospheric pressure between 700hPa to 1060hPa

7.2 Shipping and Storage Conditions

-25°C to +5°C (-13°F to 41°F) with any humidity level.

+5°C to +35°C (41°F to 95°F) at a relative humidity up to 90%.

+35°C to 70°C (95°F to 158°F) non-condensing at a water vapour pressure up to 50hPa.

8.0 DISPOSAL



When the Stand Aid has completed its life cycle and can no longer perform to its intended use safely the Stand Aid must be decommissioned by an approved Service Engineer. The following specifies the importance of correct disposal procedure including local laws and being environmentally friendly.

Please observe the local laws on recycling and respect the current laws for disposal within the community the device is being used within. If there is any uncertainty of the below guidelines, contact your local authorities to determine the proper method of disposal of potentially biohazardous parts and accessories.

The relevant components utilised in the manufacture of the device that can be recycled at the end of the device life are:

Fully recyclables:	Consideration when Recycling:
Steel frame (Frame, Upper Assembly, Legs)	Knee Pad
Castors	Leg Actuator
Initial packaging of the device (cardboard)	Lift Actuator
Metallic fixing – screws etc.	Handset + Cables
Foot Tray	Control Box
	Battery

Table 11



The product may be contaminated and has to be disinfected before recycling or disposal. See section 10.3 'Cleaning' for details of how to do this.

9.0 FAULT FINDING

If a problem arises with the Stand Aid, the Table below will hopefully assist in determining the fault and what actions you can take. If the fault cannot be found or the fault is found and the action guide does not provide a fix (e.g. – a damaged wire would need replacement), contact your local Prism Medical UK authorized dealer immediately, a service engineer will be required to repair the Stand Aid. Contact details can be found on the last page of this manual.

Fault	Action
Stand Aid doesn't turn on	<ul style="list-style-type: none"> • Check the E-Stop button on the control box has not been activated. • Check the Handset is correctly connected to the correct control box port. • Check the Handset lead is not damaged, e.g. cuts and breaks. • Check to see if the Stand Aid battery is out of charge and requires charging. • Check the Battery is located correctly on top of the control box. • Check for any damage to the battery terminals, e.g. burns and breaks. • Perform a visual check on the Control box for any damage.
Stand Aid doesn't charge	<ul style="list-style-type: none"> • Check the Charger plug is turned on at the power supply. • Check that the Charger Lead is connected correctly to the Control Box • Check charging lead for damage, e.g. cuts and breaks. • Check that the Battery is connected properly with the Control Box. • Inspect the Battery terminals for any damage, e.g. burns and breaks. • Inspect the Control Box terminals for any damage, e.g. burns and breaks
Stand Aid doesn't lift	<ul style="list-style-type: none"> • Check if the Stand Aid is plugged into Charge. (Stand Aid will not raise and lower while on charge) • Check that the Lift Actuator to Control Box cable is connected properly between the two ports. • Check the Lift Actuator to Control Box cable for damage, e.g. cuts and breaks. • Inspect the Lift Actuator for any signs of damage which could affect its function, including the Port. E.g. Large Cracks and breaks, electrical burns. • Inspect the Control Box Port for any damage. E.g. electrical burns, breaks.
Stand Aid is functioning but LCD and/or LED is inactive	<ul style="list-style-type: none"> • Check LED externally for any impact damage, e.g. cracks or breaks. • Control Box/handset is damaged
Stand Aid Leg Actuator is not responding.	<ul style="list-style-type: none"> • Check Cable between the Control Box and the Leg Actuator is connected correctly. • Check for Cable damage between the Control Box and Leg Actuator cable. • Inspect the Leg Actuator for any damage, including the cable port. E.g. large cracks and electrical burns. • Inspect the Control Box Leg Actuator Cable Port for any damage. E.g. cracks and electrical burns. • If there is no indication of any power while commanding the Powered Legs, Handset buttons could be damaged.
Stand Aid has power but does not respond to handset commands.	<ul style="list-style-type: none"> • Check the Handset is correctly connected to the Control Box. • Check that the Lift Actuator and Leg Actuator Cables are both connected to the Control Box. • Check the Handset cable is not damaged, e.g. cuts and breaks. • Check the Lift Actuator and Leg Actuator Cables for damage, e.g. cuts and breaks • Check the Control Box Handset Port for any damage. E.g. cracks and electrical burns.

Table 12

10.0 GENERAL INSPECTION, MAINTENANCE AND CLEANING

10.1 Service



No service is to be carried out on the Freeway SA180C while transferring a person to reduce the risk of injury. Service must be completed by a Prism Medical UK authorised Service Engineer. Do not attempt to service the product yourself, or warranty is void.

To ensure the safety and continued good function of your Stand Aid, routine service must be performed on your Freeway SA180C Stand Aid.

Service should be completed by a Prism Medical UK approved service engineer every 6 months to ensure the products required standard is maintained. The service history of the product should be documented each service in the Service Log at the back of this User Manual.



When the Stand Aid is serviced, the 6 month service checklist must be completed for the Freeway SA180C Stand Aid. Service Manual Document Number: 995066. Spare Parts Manual Document Number: 992066. The Service must be completed every 6 months after installation of the Stand Aid to comply with LOLER Regulations.

The Freeway SA180C Stand Aid has an expected Service Life of 10 Years.

Contact your local authorised Prism Medical UK dealer if you:

- Need more information.
- Have any questions about the use or service of your Stand Aid.
- Notice any change in the performance.
- Want to report an unexpected occurrence.
- Want to arrange a service.
- Need to ascertain necessary information for replacement parts and components.

Contact details of your local Prism Medical UK dealer are shown on the last page of this manual.

10.2 Inspection

Inspection is to be completed prior to each use by the user of the Stand Aid.



Should any of the components in the table below fail the inspection, DO NOT use the Stand Aid. Contact your local authorised dealer for service – contact details are on the last page of this manual.

Ensure all component inspections in the Table below are completed prior to each use of the Stand Aid.

Check List before Use

Component	Service/Inspection required
Generic	Visual inspection of the external of the Stand Aid. Significant damage that may affect the function of the Stand Aid along with a clear safety hazard is unacceptable.
	Check the Labelling on the Stand Aid to ensure they are all still legible, this includes the Serial Number and other important markings. If labels are not legible, then contact your local authorised dealer immediately.
	Check all main nuts and bolts to see if they are loose, If they are not tight or you have concerns, then contact your local authorised dealer immediately.
Certification	An authorised/competent service company or person will issue a test certificate after satisfactory completion of the LOLER inspection. The certificate will be valid for 6 months.
Knee Pad	Inspect the Knee Pad for damage including cuts and breaks.
	Ensure all height adjustment to the Knee Pad is moving freely and is not jamming.

Foot Tray	Ensure the Foot Grips are still applied and undamaged. Check Foot Tray for permanent deformation.
Battery Pack	The battery pack should not require maintenance other than the regular charging as detailed in the charging instructions.
Emergency Stop Button	Check the emergency stop button functionality.
Manual Emergency Lower	Check the manual emergency lower functionality.
Actuator	The actuator should not require maintenance other than checking for correct operation, listening for unusual noise and checking for any damage.
Leg Actuator	The actuator should not require maintenance other than checking for correct operation, listening for unusual noise and checking for any damage.
Upper Assembly	Always ensure that the star handles are tight before each use. Sufficiently tighten both handles in a clockwise direction to secure the upper assembly. This should be done after each adjustment and before each use to ensure the patients safety.
Castors	Check the brake functionality on each rear castor. Check all castor fixing points. Check that each castor runs free and rotates easily. Remove any build-up of hair, fluff, dust etc.
Leg Pivot Points	Check the smooth opening and closing of the legs.
Control Box	Inspect all male plugs and female sockets for correct fitting. Inspect the hand control functionality.
Sling Attachment Points	Inspect the sling looped attachments for any damage, sharp edges and excessive wear.

Table 13

10.3 Cleaning

Please follow the cleaning guidelines below on cleaning and disinfecting the Stand Aid.

10.3.1 General Cleaning



It is recommended to clean the Freeway SA180C and accessories before use by a different person, reducing the risk of cross-contamination.

The exterior of the Freeway SA180C can be cleaned using a damp soapy cloth for general cleaning duties. Please ensure the cloth is damp and not wet. Ensure the exterior of the device is dry after cleaning. Dry using a clean dry cloth.

10.3.2 Disinfecting (if necessary)

Should the Stand Aid require a more thorough clean, the use of the Actichlor™ disinfectant product (which is widely available in tablet form and used throughout the health care industry) is recommended.



Follow the manufacturer's safety instructions for the use of the cleaning product before use to ensure safe use for the operator and the patient.

Ensure the cloth is damp before the cleaning process.

Application is through a clean damp cloth applied to wipe the device down. Use in the following dilutions to ensure an effective clean:

- Actichlor™ dissolvable chlorine tablets provide a concentration of 1000 ppm of available chlorine (0.1%) per 1 tablet
- 1 tablet (1.7g formed tablet (x1)) will create a virucidal solution, diluted in 1 litre of water to provide effective means to clean a "dirty" device. This is also ideal for use after an outbreak of the Norovirus/winter vomiting and can be used as a precaution against C.Diff. It is effective against viruses, bacteria, spores, yeasts and moulds.
- The contact time against the outer components of the device should be for 5 minutes to prevent any virucidal infections without a degradation to the functionality of the device. 5 minutes is a recommended contact time. The device can withstand a longer contact period but the 5 minute recommendation as a minimum must be followed to provide an effective cleaning regime.
- Blood spills should be dealt with by an increased concentration of the solution – please refer to the instructions on the manufacturers product labelling.

Dilution chart					
Product used as	Device condition	Concentration (ppm)	Dilution qty* (l)	Tablets per 1l (0.26gal)	Contact time (minutes)
Bactericidal	Clean	200	5 (1.32gal)	1	1
	Dirty	1000	1 (0.26gal)	1	5
Yeasticidal	Clean	200	5 (1.32gal)	1	1
	Dirty	1000	1 (0.26gal)	1	5
Fungicidal	Clean	2000	1 (0.26gal)	2	15
	Dirty	5000	1 (0.26gal)	5	15
Mycrobactericidal	Clean	1000	1 (0.26gal)	1	15
	Dirty	5000	1 (0.26gal)	5	15
Virucidal	Clean	500	2 (0.53gal)	1	5
	Dirty	1000	1 (0.26gal)	1	5
Sporcidal (C.Diff)	Clean	1000	1 (0.26gal)	1	10
	-	-	-	-	-
Sporcidal	Clean	5000	1 (0.26gal)	5	10
	-	-	-	-	-

* Dilution is made with water. DO NOT dilute within any other medium.

- When diluted in water, one tablet gives 1000ppm of available chlorine.
- The concentration of the solution depends upon whether the object being cleaned is noticeably dirty (indicated in the table by "Device condition").

Table 14

Handling and storage safety precautions when using this cleaning agent:

Advice on Safe Handling



- Avoid contact with skin and eyes.
- Do not breathe dust/fumes/gas/mist/vapours/spray.
- Use only with adequate ventilation.
- Wash hands thoroughly after handling.
- Mixing this product with acid or ammonia releases chlorine gas.

Hygiene Measures

Handle in accordance with good industrial hygiene and safety practice. Remove and wash contaminated clothing before re-use. Wash face, hands and any exposed skin thoroughly after handling.

Conditions for Safe Storage



- Keep out of reach of children.
- Keep container tightly closed.
- Store in suitable labelled containers.
- Storage temperature: 0-25°C (32-77°F).

Individual Protective Measures

Hand protection: Gloves

Dissolve

Dissolve in cold water – With no agitation, 1 tablet will take approximately 10 minutes to fully dissolve in the water used.

The information above has been extracted from the Actichlor™ MSDS (Manufacturers Safety Data Sheet). For a full review of the data please follow the link below:

<http://www.nhsggc.org.uk/media/236215/msds-actichlor-plus.pdf>



11.0 WARRANTY

This guarantee does not affect or in any way limit your Statutory Rights.

1. Prism Medical UK guarantees the Freeway SA180C, supplied as new, against failure within the period of 36 months from the date of purchase by virtue of defects in material or workmanship. (Excluding Batteries)
2. The liability of Prism Medical UK under terms of this guarantee shall be limited to the replacement or the defective part(s) to the sales distributor, dealer, agent, person or entity which purchased the equipment from Prism Medical UK. In no event shall Prism Medical UK incur liability for any consequential or unforeseeable losses.
3. This equipment guarantee shall be void if the equipment is not serviced by Prism Medical UK or its authorized agents, in accordance with manufacturer's recommendations, or if any unauthorized persons carry out work on the equipment.
4. This guarantee does not apply to failure attributable to normal wear and tear, damage by natural forces, user neglect or misuse or to deliberate destruction.
5. Do not attempt to service the product yourself, or warranty is void.
6. Removal of the warranty label on the side of the control box will result in the Warranty becoming void.

12.0 SERVICE RECORD HISTORY

Complete this section after each service, repair inspection and/or maintenance.

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____	Time: _____
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other	
Completed by: (printed name). (signature)	
Company:	
Remarks & Action Taken:	
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)	

Date: _____	Time: _____
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other	
Completed by: (printed name). (signature)	
Company:	
Remarks & Action Taken:	
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)	

Date: _____	Time: _____
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other	
Completed by: (printed name). (signature)	
Company:	
Remarks & Action Taken:	
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)	

Date: _____	Time: _____
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other	
Completed by: (printed name). (signature)	
Company:	
Remarks & Action Taken:	
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)	

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by:		(printed name). (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by:		(printed name). (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by:		(printed name). (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by:		(printed name). (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

Date: _____		Time: _____	
Service Type: <input type="checkbox"/> Periodic inspection <input type="checkbox"/> Monthly inspection <input type="checkbox"/> 6-month inspection <input type="checkbox"/> Repair <input type="checkbox"/> Yearly inspection <input type="checkbox"/> Other			
Completed by: (signature)	
Company:			
Remarks & Action Taken:			
Device left in a safe usable condition: YES. <input type="checkbox"/> NO <input type="checkbox"/> (if "NO" explain in remarks the action taken)			

User notes:

Dealer/service contact details:

Manufacturer contact details:

Prism Medical UK

Address: Unit 1 • Tir Llwyd Industrial Estate • St Asaph Avenue • Kinmel Bay • Conwy • LL18 5JZ

Telephone Number: 01924 840100

Disclaimer

While every effort has been made to ensure the accuracy of information contained in this user manual, no liability can be accepted by Prism Medical UK for any errors or omissions.

Prism Medical UK operates a policy of continuous improvement. Specifications and other data are subject to change without notice.



freeway

Unit 1 • Tir Llwyd Industrial Estate • St Asaph Avenue •
Kinmel Bay • Conwy • LL18 5JZ



119129